

	Statim2000 G4	Midmark M3 UltraFast	Comments
Unit Size (L x W x H)	19.5" L x 16.25" W x 5.9" H	22" L x 17.8 W" x 6.9" H	
Chamber Size	11" (49.5 cm)L x 7"(41.5cm) W x 1.4"(15cm) H	12.1" (30.7 cm) L x 7.6" (19.3 cm) W x 1.6" (4.1 cm) H	10% more capacity
Sterilization Chamber Volume	.43 gal	.49 gal	
Unit Weight	46 lbs – no information on shipping weight available.	71 lbs (36.3 kg) / shipping weight 80 lbs	M3 is UPS shippable
Electrical Requirements	100V, 50 /60 Hz, 11A 110V, 50/60 Hz, 11A 220-240V, 50/60 Hz, 6A	115 V, 50/60 Hz, 12A 230V, 50/60 Hz, 6A	M3 is available in both 115 V and 230 V (separate models)
Cycle Times:			
Unwrapped	6 minutes 60 minute dry (Total of 66 minutes)	6 minute cycle 25 Minute dry (Total of 31 minutes)	M3 uses unique surface heaters on the chamber top and bottom to produce heated drying. Cycle time includes: warm up & sterilization
Wrapped / Pouches	14 min cycle 60 min dry (Total of 74 minutes)	10.5 min cycle 30 min dry (Total of 40.5 minutes)	M3 uses unique surface heaters on the chamber top and bottom to produce heated drying Cycle time includes: warm up & sterilization
Low Temp/Rubber/Plastic	20 min cycle 60 min dry (Total of 80 minutes)	24.5 min cycle 50 min dry (Total of 74.5 minutes)	M3 uses unique surface heaters on the chamber top and bottom to produce heated drying Cycle time includes: warm up & sterilization
Dry	60 min dry cycle non-programmable	30 minute dry cycle 20-60 min programmable	M3 uses unique surface heaters on the chamber top and bottom to produce heated drying
Maintenance	door gasket replacement 500 cycles or 6 months, whichever comes first	door gasket replacement 3years or 2,500 cycles	M3 door gasket can be changed in 3 minutes
Maintenance	Bacteria Retentive Filter & Air Filter – every 500 cycles or 6 months	Gasket every 2 years or 2,500 cycles. No air filter	